
User's Manual
Bedienerhandbuch
Manual Utilisateur
取扱説明書
用户手册

MD-N2M5B Retractable Sensor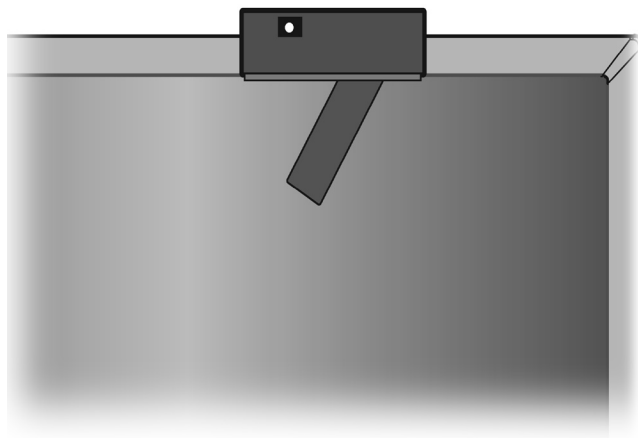

Index

Safety Instruction.....	English.1
Contents.....	English.2
Components.....	English.3
Installation.....	English.4
Troubleshooting.....	English.8
Specifications.....	English.8
Sicherheitsvorkehrungen.....	Deutsch.1
Inhalt der Verpackung.....	Deutsch.2
Bedienelemente.....	Deutsch.3
Installation.....	Deutsch.4
Fehlerbehebung.....	Deutsch.8
Technische Daten.....	Deutsch.8
Consignes de sécurité.....	Français.1
Sommaire.....	Français.2
Composants.....	Français.3
Installation.....	Français.4
Résolution des problèmes.....	Français.8
Spécifications.....	Français.8
安全のために必ず守ること.....	日本語-1
付属品.....	日本語-2
各部の名称.....	日本語-3
取り付け方法.....	日本語-4
キャリブレーション.....	日本語-6
困ったときには.....	日本語-8
仕様.....	日本語-8

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Français

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Safety Instructions

- Do not open this unit. There are no user serviceable parts inside and opening or removing covers may expose you to electric shock hazards or other risks. Refer all servicing to qualified service personnel.
- Do not insert objects of any kind into the cabinet slots, as they may come in contact with dangerous voltage points. Harmful or fatal electric shock, fire, or equipment failure may result.
- To discharge static electricity that may have accumulated in the body, touch a grounded metal object before using this unit.

Immediately disconnect the unit and refer servicing to qualified service personnel under the following conditions:

- If liquid has been spilled onto the unit.
- If the unit has been dropped or has had the cabinet damaged.

For further safety precautions and information, please refer to the user's manual that was included with LCD monitor.

Cleaning the Unit

- Unplug the cable.
- Gently wipe the unit with a soft cloth.


NOTE: Many plastics are used on the surface of the unit. DO NOT clean with benzene, thinner, alkaline detergent, alcoholic system detergent, glass cleaner, wax, polish cleaner, soap powder, or insecticide. Do not touch rubber or vinyl to the cabinet for a long time. These types of fluids and fabrics can cause the paint to deteriorate, crack or peel.

Contents

Your MD-N2M5B Retractable Sensor box should contain the following:

- MD-N2M5B Retractable Sensor
- Adhesive Foam Spacer - Thin
- USB cable (2.5m)
- Adhesive Foam Spacer - Thick
- User's Manual

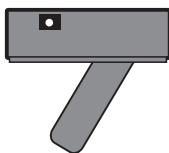MD-N2M5B Sensor

USB Cable (2.5m)


Adhesive Foam Spacer - Thin

For use with: MD21GS-3MP, MD211GS3P, MD21GS-2MP, MD211GS2P,
MD21C-2MP, MD211C2P, MD19C-1MP, MD191C1P,
LCD2180UX, RDT211H, LCD1980SXi, RDT191H

Adhesive Foam Spacer - Thick

For use with: LCD1980FXi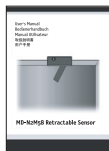


User's Manual

****Remember to save your original box and packing material for transportation or shipping.***


Components

English

Front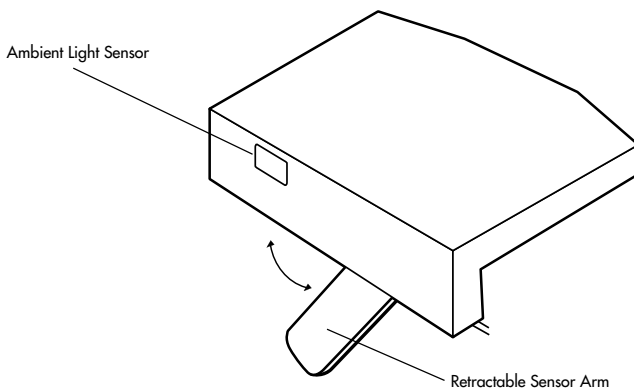

Back


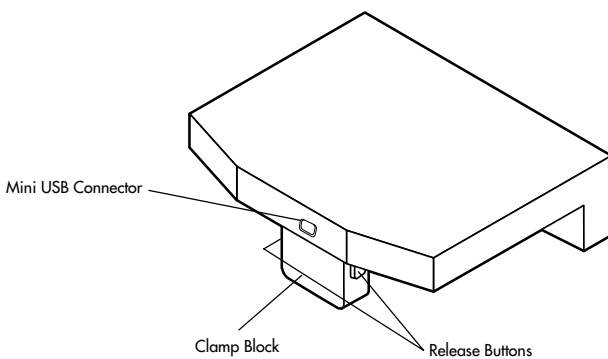
English.3

Installation

Attaching MD-N2M5B to Display

1. Press the release buttons on the sides of the Clamp Block and slide the Clamp Block to the widest position (**Figure 1**).
2. Depending on the type of monitor being used, it may be necessary to use one of the included Foam Spacer strips in order for the sensor to properly fit the display. To attach the Foam Spacer, remove the strips covering the adhesive press the spacer into place on the bottom of the sensor. The side of the spacer should be flush against the front of the sensor. Make sure the spacer is the correct one for use with your monitor type* (**Figure 2**).

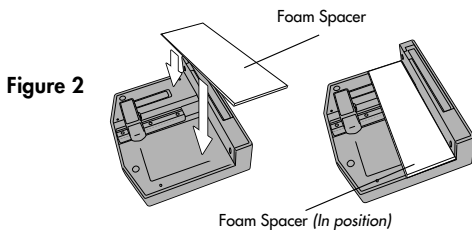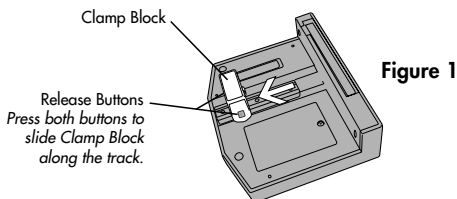
* The thin foam spacer is for use with the following models: MD21GS-3MP, MD211GS3P, MD21GS-2MP, MD211GS2P, MD21C-2MP, MD211C2P, MD19C-1MP, MD191C1P, LCD2180UX, RDT211H, LCD1980SXi, RDT191H
The thick foam spacer is for use with the following models: LCD1980Fxi

English.4


3. Set the MD-N2M5B on top of the display, with the Clamp Block hanging behind the monitor (**Figure 3, Figure 5, Figure 6**).
4. Press the Release Buttons and slide the Clamp Block forward until it is snug against the display.
5. Connect the included USB cable. The "B" (mini) connector on the USB cable attaches to the sensor unit. The "A" (USB) connector on the USB cable attaches to the computer or to a powered USB hub.

NOTE: The USB cable must be connected to a USB hub that is powered externally or to a USB port on the computer. The unit will not function without a powered USB connection.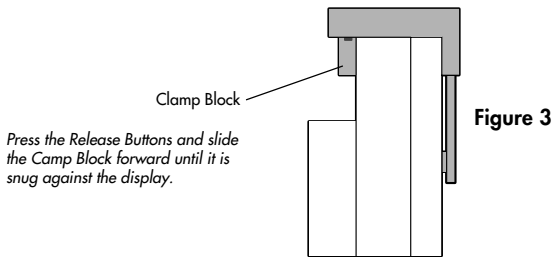

"A" (USB) connector -
Attaches to USB hub or to PC

Figure 4

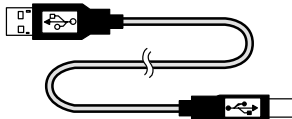"B" (mini) connector -
Attaches to Retractable Sensor

Calibrating MD-N2M5B

1. For proper operation, the Retractable Sensor needs to be correlated to a reference point in the centre of the display.
2. Plug sensors into the PC or into an externally powered USB hub that is connected to the PC. Make sure that all the drivers for the sensors are loaded into the PC. All sensors need to be plugged into the PC before the GammaComp MD software is launched.
3. Start the GammaComp MD software. For detailed information regarding use of this software, please refer to the User's Guide included with the GammaComp MD software.
4. When the GammaComp MD software is first launched, an initialisation screen will appear on the first compatible display. Each retractable sensor will have to be matched up with the correct display. In the "Optional Number 2" field of the Setup Display Interface enter "1". Click "OK". Then the target Retractable Sensor will activate. If a sensor other than the target sensor operates, click "CANCEL" and enter a different number (i.e. 2) in the Setup Display Interface. Do this until the correct sensor is activated. Once the correct sensor is linked with the display, the initialisation screen will then appear on the next compatible monitor. This procedure will be followed for each display.
5. Choose the Reference Sensor (i.e. Gretag/Macbeth Eye-One Display Sensor) that will be used to correlate the retractable sensor.
6. Right-click the "MD" icon in the task tray, and a popup menu will appear, then select "Open Display Maintenance".

English.6

7. Select the Reference Sensor (i.e. EyeOne Display) in "Sensor Name".
8. Click "Calibration for MD-N2M5B (Retractable)". The correlation of the MD-N2M5B Retractable Sensor to the Reference Sensor will begin. When instructed, place the Reference Sensor where indicated on the display (**Figures 5 and 6**). Click "OK".
NOTE: This procedure will take a few minutes to execute.

NOTE: After the MD-N2M5B Sensor is calibrated, it should not be repositioned.

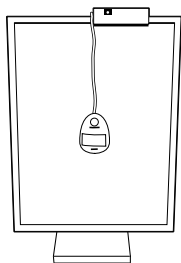
Figure 5

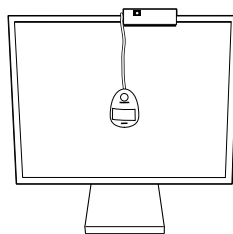
Figure 6

Troubleshooting

If the sensor arm does not respond when the GammaComp MD software is started:

- Make sure that the USB cable is properly connected. The USB cable must be connected to a powered USB hub or directly to a computer to function properly.
- Make sure that attached sensor unit is level.
- Make sure that the GammaComp MD software is a version that is compatible with the sensor.
- Make sure that the sensor is connected to a display that is supported by the software.
- Make sure that all sensors are properly connected before the GammaComp MD software is launched.
- When initialising sensor(s), make sure that in the "Optional #2" field of the initialisation dialog box, a number other than "0" is specified.

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Current Rating: 500mA (max)

Environmental Considerations

Recommended Temperature: 20°C to 25°C / 68°F to 77°F

Operating Temperature: 5°C to 40°C / 41°F to 104°F

Humidity: 30% to 80% (without condensation)

Storage Temperature: -20°C to 60°C / -4°F to 140°F

Humidity (in package): 10% to 85% (without condensation)

English.8

DECLARATION OF CONFORMITY

For USA

This device complies with Part 15 of FCC Rules. Operation is subject to the following two conditions.
(1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

U.S. Responsible Party:	NEC Display Solutions of America, Inc.
Address:	500 Park Blvd, Suite 1100 Itasca, Illinois 60143
Tel. No.:	(630) 467-3000

Type of Product: Retractable sensor
Equipment Classification: Class B Peripheral
Model: MD-N2M5B

*We hereby declare that the equipment specified above
conforms to the technical standards as specified in the FCC Rules.*

FCC Information

- Use the attached specified cables with the MD-N2M5B colour display so as not to interfere with radio and television reception.
(A) Please use the supplied USB cable.
Use of other cables and adapters may cause interference with radio and television reception.
- This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:
 - Reorient or relocate the receiving antenna.
 - Increase the separation between the equipment and receiver.
 - Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
 - Consult your dealer or an experienced radio/TV technician for help.

If necessary, the user should contact the dealer or an experienced radio/television technician for additional suggestions. The user may find the following booklet, prepared by the Federal Communications Commission, helpful: "How to Identify and Resolve Radio-TV Interference Problems." This booklet is available from the U.S. Government Printing Office, Washington, D.C., 20402, Stock No. 004-000-00345-4.

For Europe

Declaration of the Manufacturer

We hereby certify that the retractable sensor MD-N2M5B is in compliance with Council Directive 89/336/EEC:

- EN 55022
- EN 55024
- IEC61000-4-2
- IEC61000-4-3
- IEC61000-4-4
- IEC61000-4-5
- IEC61000-4-6
- IEC61000-4-8

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NEC Display Solutions, Ltd.